DANSK

SVENSKA

SUOMALAINEN

NORSK

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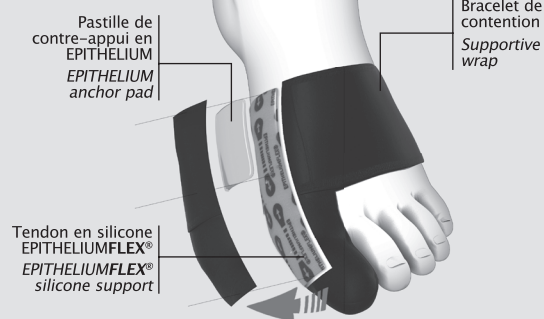
PATENTED TECHNOLOGY

ORTHÈSE HALLUX VALGUS EPI-THELIUMFLEX® 02

HALLUX VALGUS BRACE / HALLUX VALGUS SCHIENE / ORTESI PER ALLUCE VALGO / ORTESE HALLUX VALGUS / ORTHESE HALLUX VALGUS / ORTESIS HALLUX VALGUS / HALLUX VALGUS ORTESE / HALLUX VALGUS-ORTOS / HALLUX VALGUS-ORTEESI / HALLUX VALGUS-ORTOSE / ΟΡΘΩΤΙΚΟ ΒΟΗΘΗΜΑ ΒΛΑΙΣΟΥ ΜΕΓΑ ΔΑΚΤΥΛΟΥ

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
**HALLUX VALGUS BRACE EPIITHELIUMFLEX® 02**

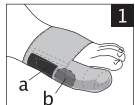FUNCTIONS:
This flexible hallux valgus brace helps to realign the big toe, limit pronation and thereby optimise propulsion. Its EPIITHELIUMFLEX® external support absorbs pressure and friction on the exostosis («bunion»). Finally, its supportive metatarsal wrap helps to limit crushing and flattening of the forefoot.

INSTRUCTIONS FOR USE:
• First of all, make sure that the big toe can still move sideways when manipulated. Otherwise, do not use this product.
• Do not use this product at night.
• The brace can be worn on either the right foot or the left foot.

FITTING INSTRUCTIONS:
• Slip the front of the forefoot into the wrap.
• Then slip the brace over the big toe.
• Position the Epithelium™ pad (a) behind the bunion (b). (See diagram 1).


ADJUSTMENT:
• When using for the first time, do not try to completely realign the big toe.
• Over the course of the first 10 uses, gradually realign the big toe by pulling the brace backwards. (See diagram 2).
The brace may be worn daily for optimum results.

WASHING INSTRUCTIONS:
• Machine wash at 40°C or hand wash (without rubbing).
• Leave to dry naturally. Do not iron.

CONTRAINDICATIONS:
• Big toe not mobile.
• The product has undergone all the regulatory tests required by ISO standard 10993-1 relative to products that come into contact with the skin. However, if any skin reactions are observed, stop using the product immediately and contact MILLET Innovation.

• Do not use on an unprotected wound.
• Individuals with arterial disease, diabetes, polyneuritis, neuropathy of the lower limbs or varicose ulcers: never use the product over an open wound or diabetic foot ulcer. Monitor the foot during use and ask your doctor or podiatrist for advice.

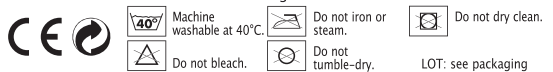COMPOSITION: 52% silicone, 48% polyamide/elastane fabric.
MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use.

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