

ENGLISH

epitact.

## Light legs THERAPEUTIC INSOLES

*Stimulates blood circulation with each step.*

---

### TRIPLE ACTION:

- Act against heavy legs
- Relieve painful legs
- Soothe feelings of swelling

**INDICATIONS:** The use of the EPITACT®'s LIGHT LEGS Insoles is recommended for people with circulatory disorders of the lower limbs, venous insufficiency or varicose veins.

**INSTRUCTIONS FOR USE:** Thin and easy to put on, the insoles are compatible with all closed shoes up to 1 inch heel. Do not use with or instead of your orthopaedic insoles. They are compatible with but not a substitute for medication and/or medical compression devices. If necessary, adapt the insoles to your shoe size. If they move in your shoes, use the holding pads supplied in the box.

**CONTRA-INDICATIONS:** Do not use on injured skin. In case of arteritis, phlebitis, diabetes, neuropathy of the lower limbs or allergy to one of the components: seek advice from a healthcare professional and watch your foot during use.

**CARE INSTRUCTIONS:** Remove the insoles from your shoes after use. Wash them regularly by hand with soap and water. We recommend washing them after each use. If signs of wear (tears) appear after a period of use, replace the insoles.

**COMPOSITION :** 80,5% SEBS, 19,5% PP.

In case of pain or any other unusual reaction, contact your doctor. Any major incidents related to this product must be reported to the Millet Innovation laboratory and the relevant health authority (MHRA). Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. CE initial marking: 01/2023.



50°C  
Store away from UV light  
and at a temperature  
below 50°C.



Product of MILLET Innovation  
research



Hand  
washable.



See  
packaging.



Medical  
device