

DIGITOP with EPITHELIUM 26®

INDICATIONS

Prevents and soothes pain at the tip of the toes: hard corn, blue nail, ingrown toenail, etc. Worn daily, they help to eliminate corns.

CONTRA-INDICATIONS

Do not use on injured skin. In case of arteritis, diabetes, lower limb neuropathy or ulcers, seek advice from your doctor or podiatrist before use and then regularly monitor your foot.

TIPS FOR USE

- For easy application, position the device upside down on the tip of your finger or toe and unroll it gradually over the finger or toe.
- Do not shorten the Digitop as this will adversely affect its durability.
- To be removed every evening to avoid risks of maceration.

FEATURES

Excessive rubbing and pressure cause hard skin. The Digitop contains the patented EPITHELIUM 26® silicone gel, that protects the toe, relieves pain immediately and helps corns to disappear.

DESIGN

The EPITHELIUM 26® gel is inserted between two layers of soft fabric. Very slim & comfortable, it can be worn in any shoe.

CLEANING & LIFESPAN

Great resistance to wear and tear.


Hand washable.


Washable and reusable for one or two months.


COMPOSITION: 62 % silicone, 38 % polyamide/elastane fabric.


The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2001.





 Hand washable.

 Do not iron or steam.

 Do not dry clean.

 Do not bleach.

 Do not tumble-dry.

 See packaging.

Product of MILLET Innovation research.

 MILLET Innovation ZA Champgrand
26270 Loriol/Drôme - FRANCE

MILLET
INNOVATION

www.epitact.co.uk

N10076EN-K
09/2018