

FLEXIBLE THUMB BRACE with EPITHELIUMFLEX®

INDICATIONS

Developed to relieve joint pain associated with rhizarthrosis or osteoarthritis of the thumb, this flexible thumb brace allows the movement of everyday life while limiting traumatic micro-movements.

CONTRA-INDICATIONS

Do not use on injured skin. In the event of severe oedema: look closely at the splint during use. Remove the splint if it is too tight and impedes circulation. When in doubt seek advice from your GP or your pharmacist.

PRECAUTIONS FOR USE

This product is a day use splint. Two versions are available: right hand and left hand splints (stitching is inside the hand splint). Do not use near a flame or in incandescent body.

FEATURES

It limits the traumatic micro-movements (movement of the hands in space by walking, speaking, ...), keeping the thumb in the rest position and preserves heat at the joint and relieves joint arthrosis pain at the root of the thumb by shock-absorbing.

DESIGN

Flexible, thin and discreet, this brace allows you to retain the full functionality of your hand: you can drive, write, use your phone... and wear your watch or jewellery!

ADVICE

The flexible EPITACT® brace can be used alongside with the EPITACT® rigid brace that is better suited for night-time use.

CLEANING & LIFESPAN

Wash in a machine at 30°C / 86°F or by hand with soap and water (without rubbing) and leave to dry naturally. Do not iron. Reusable for several months.


COMPOSITION: 86% polyamide/elastane fabric, 14% silicone.


The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.


Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2013.




 Machine washable at 30°C / 86°F.

 Do not iron or steam.

 Do not dry clean.

 Do not bleach.

 Do not tumble-dry.

 See packaging.

Product of MILLET Innovation research.

 MILLET Innovation ZA Champgrand
26270 Lorioi/Drôme - FRANCE

MILLET
INNOVATION

www.epitact.co.uk

N11441EN-E
09/2018